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All instruments have been developed and constructed for their specific use. Improper use can lead to damage to the tissue, premature wear, destruction of the instruments, danger to the user, patient or third parties.

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#### User Information for rotating dental and surgical Instruments

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– Use of improper instruments can cause insufficient preparation results.

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– Excessive speeds while using instruments with overly large heads endanger the stability of the instruments.

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– Excessive speeds applied near to the dental pulp can endanger its vitality. The same can be the result when using an inadequate supply of water spray.

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– Observe permissible speeds indicated on packaging.

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– The handpiece must be in proper working condition. The use of an adequate supply of water spray of a minimum of 50 ml/min on the working part is necessary in the 1500 min<sup>-1</sup> speed range to ensure reliable cooling of the material being worked on.

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– Additional external cooling is needed for instruments with a total length of > 19 mm and a head diameter of 1.8 mm (ISO -018).

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– Moistening of the surfaces to be prepared must not be impaired by an incorrect suction system. Deflection of the coolant jet can also cause heat related damage.

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– The recommended applied contact pressure range is 0.3 – 2 N (30 – 200 p) depending on the kind of preparation.

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– Avoid tilting, levering, or blocking the instruments up by using too much contact pressure (increased danger of fracture).

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– Insert the instruments as deeply as possible without excessive use of force into the chuck and check to ensure that they are secure.

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– Extremely tapered instruments will wear faster at the tip due to the smaller size. To maintain the longevity of these instruments, careful preparation with less contact pressure is important.

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**Storage and Keeping Conditions:**

- In dry conditions and protected against contaminants. Protect instruments in general against chemicals, acids, heat and extreme temperature variations. Do not use polishers after expiration date indicated.
- **Observe guidelines from Robert-Koch-Institut - ch. E3, (Hygiene Requirements in Dentistry) or the valid legal and hygiene regulations in the respective country!**
- **Steel instruments** are unsuitable for both mechanical preparation and the steam steriliser. They should be replaced by corresponding tungsten-carbide instruments.
- **Single-use products** (identified on the packaging with ☒) are not approved for reuse. Safe use cannot be guaranteed if these products are reused, as there is the risk of infection and/or the products are no longer safe to use.
- The end of the product's service life is generally determined by wear and damage due to use.
- **Colour-anodised aluminium units** (e.g. Bur block 40500 to 40580 and Retopin mandrel) lose their colour during use of standard cleaning procedures and in a cleaning and disinfection machine. Cleaning and disinfectant agents, which are specifically designed for these materials should be used during preparation.

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**Recommendations for use**

**Polishers / Brushes**

- Apply low contact pressure in order to minimize heat generation.
- Polish with circling movements.
- In order to achieve a high-shine polish, for multiple step polishing procedures all polishers are to be used in the indicated sequence.
- Eye protection is recommended.

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**Endodontic instruments**

- Use instruments ascending in size.
- Don't omit a size.
- As small instruments are more sensitive to strain they should be used only once.
- Large sizes are not suited for curved canals = risk of breakage and perforation.
- Please check the instruments after each use for wear and deformation.
- Protect patients against aspiration. Use rubberdam, secure instruments with safety chain or cord.
- Frequent removal of dentine chips eliminates clogging of the blades. Rinse instruments in 2.5% NaOCl-solution after each use.
- Instruments which are used for the preparation of extremely curved canals should be used only once.

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**Hand instruments**

**Nerve broaches 9107 (extirpation needle)**

- Push into canal and pull out with a rotary motion of 180°.

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**K-burs 171**

- Reaming motion in clockwise direction, max. 90°.

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**K-files 173**

- Filing motion in clockwise direction, max. 45°. Usual methods, step back, step down, balanced force technique.

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**Hedstroem-files 174**

- Pushing and pulling motion without rotation. Rotational motion would increase the risk of jamming of the sharp blades. In case the canal has been enlarged with K-burs or K-files, the Hedstroem file used subsequently should be of smaller or same size.

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– Only to be used in the yellow contra-angle 90°, 450 - 800 min-1.

**Gates**

– Green contra-angle, 450 - 800 min-1. For preparation of the coronal portion of the root canal, before or after the use of files or K-burs.

**Peeso**

– Green contra-angle, 800 - max. 1200 min-1. Only to be used for opening the canal entrance and for reaming the coronal portion.

**Root Canal reamer**

– Contra angle, green 800 - max. 1200 min-1. For preparation of the root canal entrance and enlarging the coronal portion.

**Instruments for root filling**

**Root fillers Typ „L“ 178 / 178S**

– Green contra-angle. Carefully insert root filler into filling material while the contra-angle is switched off. Stop just before reaching the apex, then start the contra-angle and rotate the filling material in at a speed of max. 800 min-1 and slowly pull out the instrument.

**NT Endo-Files**

– Adhere strictly to the maximum permissible speed of 300 r.p.m. as exceeding this speed may cause fracturing.

– Use an endomotor-handpiece with controllable torque.

– Maximum torque for .04 taper version: 1.5 - 1.8 Ncm. Maximum torque for .06 taper version: 2.4 - 2.9 Ncm.

**Preparation sequence (crown-down technique)**

– Prepare straight coronal canal access region using file No. 10.

– Create the guide path using a No. 010/015 K manual file and determine the working length (WL).

– Then prepare to approx. 3 mm short of the working length (WL) using a No. 06 and No. 04 file.

– Apical finishing then follows using a No. 06 and No. 04 file.

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**Sintered diamonds**

**For cleaning, have to be dressed with the cleaning stone Art. No. 9920 from time to time:**

- Water cleaning stone to minimize dust formation and to prevent damage to the abrasive.
- Short-time grinding on the cleaning stone at 8.000 rpm
- To exclude any damage to the instrument shank, avoid contact of the cleaning stone with the shank during the grinding procedure.

**Sharpening of DSB abrasives:**

- Sharpening of blunt Sintered diamonds is carried out in the same way as the cleaning procedure but for a longer period of time. Thus, the bonding material containing used, blunt grains is removed and new, sharp grains are exposed.

**Cleaning of cleaning stone**

- It is recommended to subsequently clean the cleaning stone in the ultrasonic bath during 1 - 2 minutes.

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